
USER GUIDE

Wax pot warmer, ref. EPILCAN (WKE007)

Please read this document carefully before using the device.
Keep it in an accessible place so that you can read it at any time.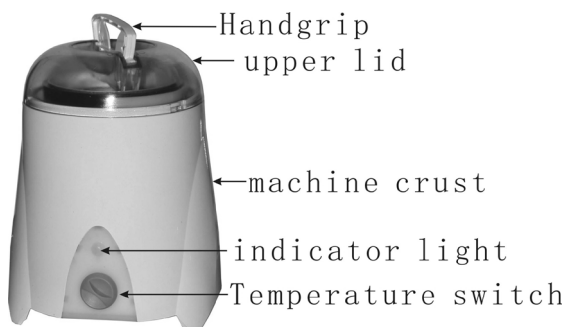

Operation

1. Put the wax into the inner housing and connect the machine to the power.
2. Please read this document carefully before using the device. Keep it in an accessible place so that you can read it at any time.
3. When the indicator light turns into green, the machine starts heating.
4. Then, adjust the temperature accordingly to the patient comfort.
5. Before starting to remove the hair, clean the grease and cosmetic that may remain on the skin with cotton that has been previously soaked in an astringent lotion. You can also use a toilet powder that absorbs moisture and grease. This will make the hair adhere to the wax more easily.
6. Daub the soft wax following the direction the hair growth.
7. After being heated, the wax should be applied with a wooden spatula on the skin to be treated. The thinner the wax layer is, the easier the hair removal becomes.
8. Do not surpass the paper band surface.
9. Hold with one hand the paper band tightly, and use the other to press it against the skin.
10. Pull the paper band quickly in the opposite direction the

hair growth.

11. After use, apply some smoothing lotion to clean the skin.

Remarks and recommendations

- Do not pour the wax after treatment if it is still liquid. Turn the machine off first and let the wax cool down. Turn it on again and when the wax starts melting at the bottom, the wax block will be easier to remove.
- Do not turn the machine on when the pot is empty.
- When you add wax, move or clean the machine, turn it off.
- Do not directly touch the pot with your hands. Take it out with the provided tool to avoid getting burnt.
- This device is intended for professional use.
- Use alcohols or other wax removal lotions to clean the machine.
- Turn the machine off before cleaning it.
- Do not move the machine when it is heating or when the wax temperature is too high. The temperature may reach the 115⁰C while working.
- Do not spill the wax. Clean it up if it gets accidentally spilt, otherwise it will become difficult to remove it.
- If the power cord is damaged, get the component at the manufacturer or at an authorised technical service.
- Do not use it on damaged or sensitive areas of the skin.
- Do not pour the used wax into the outer pot of the machine.

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