Imbustatrice per strumenti

MANUALE D'USO

IT

USER'S MANUAL

EN

MANUEL D'INSTRUCTIONS

FR

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DE

IT

Manuale d'uso

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Dear Customer,

Thank you for choosing our **Instrument Packer** model; we hope and trust that you will be very satisfied with your purchase.

The **Instrument Packer** model is designed to seal instruments used for any kind of medical or cosmetic procedures in special aseptic pouches, to prevent bacteria infections.

Copyright

All rights reserved. Reproduction of any part of this manual in any form whatsoever, without the prior written permission of XANITALIA srl is strictly prohibited. The contents of this manual may be modified without prior notice. While all care has been taken to ensure the correct nature and completeness of the documentation contained in this manual, XANITALIA srl accepts no liability for its use. This condition also applies to all companies involved in the creation and production of this manual.

Published by: XANITALIA srl, via Divisione Acqui, 53 - 61122 Pesaro - ITALIA

Important safety instructions

Read these instructions with care before using the Instrument Packer.

Before using this product, read these instructions with care. The manufacturer will accept no liability for any damage to people or property that derives from failure to respect the warnings and instructions contained in this booklet.

The guidelines given in this booklet explain the first-time operation, use and correct maintenance of the product. Follow these indications to guarantee the durability and electrical and mechanical reliability of the product. Keep this instruction booklet for later reference.

Never use the product for anything other than its intended use, which is stated in this booklet.

After removing the product from its packaging, check that it is complete and in the event of any doubt, contact a qualified technician. Keep packaging out of the reach of children or incapable persons.

If the product should fall or receive any hard knocks, it must immediately be checked by the manufacturer or by an authorised Technical Assistance Centre.

The use of any electrical appliance requires that several basic rules be observed. These include:

- never touching the appliance when hands are wet or humid;
- never touching the appliance when feet are bare;
- never leaving the appliance within reach of children or incapable persons.

Connect the product to the power supply/plug socket only if:

- the electrical data for the main supply corresponds to those on the appliance rating plate;
- the capacity of the power supply installation/plug socket is sufficient for the maximum power of the appliance.

If this is not the case, contact a qualified technician.

In the event that the appliance plug is not compatible with the plug socket, this latter must be replaced with a suitable socket by a qualified technician.

Never use worn or damaged power cables. If the power cable becomes damaged in any way, it must be replaced by the manufacturer or by an authorised Technical Assistance Centre or in any

case, by similarly qualified technicians, in order to prevent any risks.

In general, the use of adapters, multiple sockets and/or extension cables, is not advised. In the event that it is necessary to use such items, only use single or multiple adapters and cables that comply with the safety regulations in force. Take care not to exceed the limit capacity for the current as shown on the adapter or cable and the maximum power level shown on the multiple adapter.

Do not leave the appliance switched on when not in use: switch off the main switch.

Always unplug the from the mains supply before carrying out any cleaning or maintenance operations.

In the event of incorrect operation and/or failure, switch off the appliance and contact the manufacturer or an authorised Technical Assistance Centre immediately. Should repairs become necessary, always request that original replacement parts be used.

When no longer required, switch off the appliance and unplug it from the main supply. Then store it out of reach of children or incapable persons.

The electrical system to which the product is connected must comply with the relevant legal requirements for domestic wiring systems.

The cleaning and servicing operations assigned to the user must not be carried out by children unless they are over 8 years of age and are properly supervised. Keep the device and its power cable out of reach of children less than 8 years of age. Children should not play with the appliance.

CAUTION: DO NOT USE THIS DEVICE NEAR TO BATHS, SINKS OR OTHER RECEPTACLES WHICH CONTAIN WATER.

Special Warnings

This appliance has been manufactured with good materials and workmanship.

The appliance must be connected to a properly earthed socket.




Do not expose the appliance to atmospheric agents.

The device is for use only for packing surgical instruments

Do not soak the appliance or any of its parts in water or other liquids (unless otherwise specified in the section entitled "Cleaning").

Do not use the appliance in the presence of flammable fumes or substances such as alcohol, insecticides, petrol, etc.

Assembling

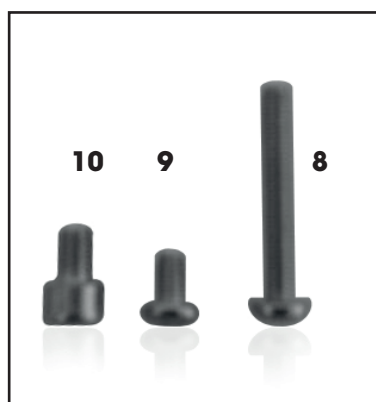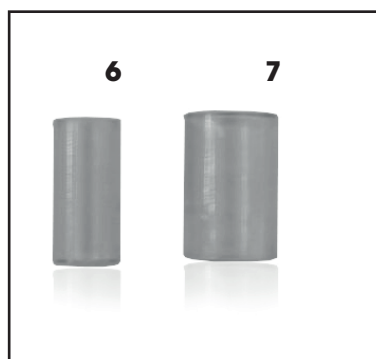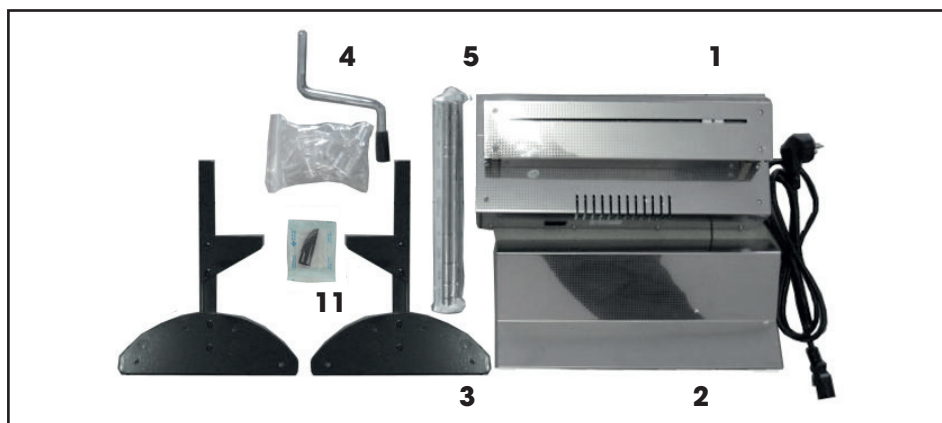- 1-Stand
- 2-Slide platform
- 3-Rack
- 4-Handle
- 5-Tubular reel supports
- 6-Small tubular support roller
- 7-Large tubular support roller
- 8-Rack fixing screw
- 9-Small tubular support fixing screw
- 10-Large tubular support fixing screw
- 11-Spare blade

STEP 1




Place the slide platform (2) on the stand (1) using the inserts on the stand.

STEP 2

Screw the rack (3) onto the stand (1) using the fixing screws (8).

STEP 3

Screw down the knob (11) to adjust the position of the blade on the stand (1).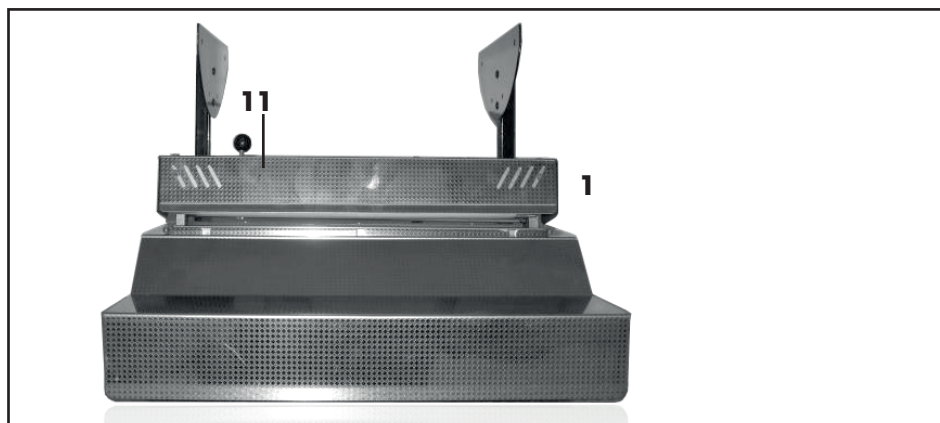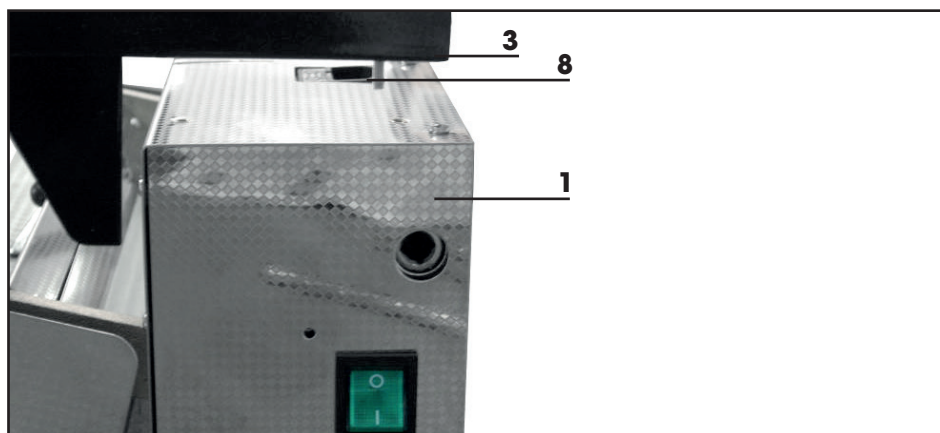
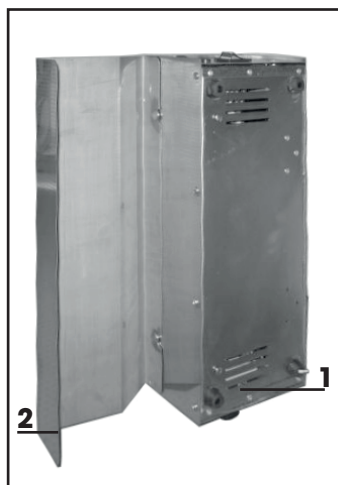



STEP 4

Fit the handle (4) beside the switch.


STEP 5

Fit the rollers (6-7) into the tubular supports (5). Then screw the tubular supports onto the rack as shown. Place the large tubular supports at the top (with the screws 10) and the small ones at the bottom (with the screws 9).

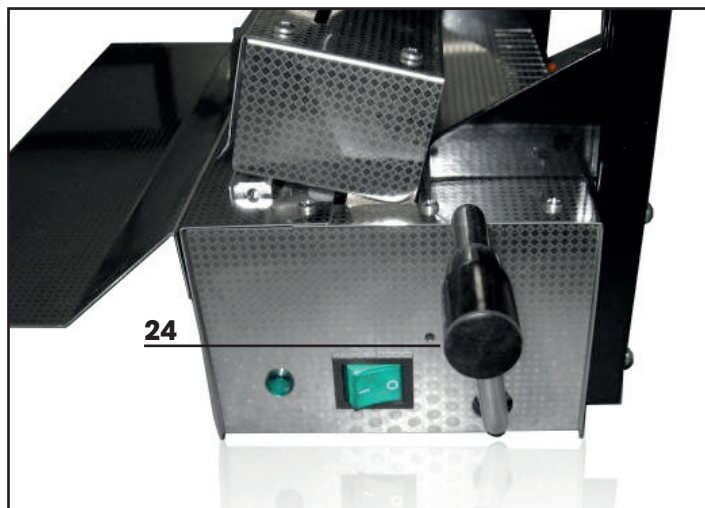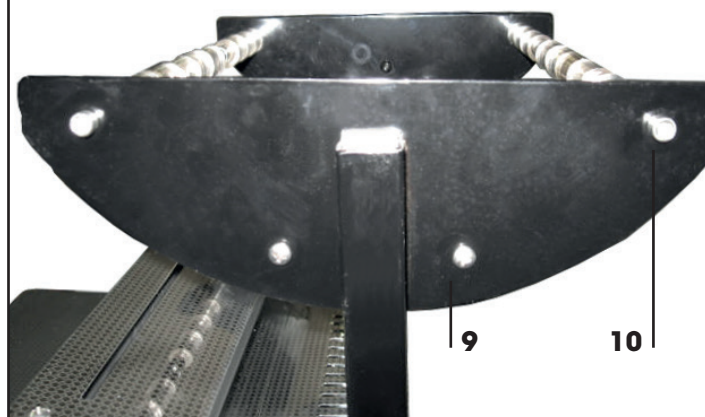**ROLLERS IN THE TUBES**

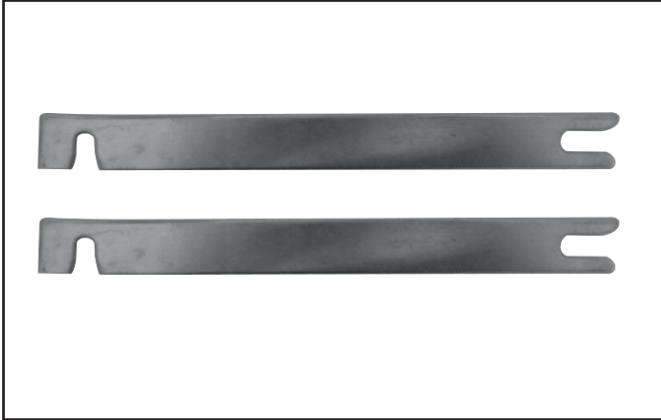**STEP 12**

Reel separator brackets.

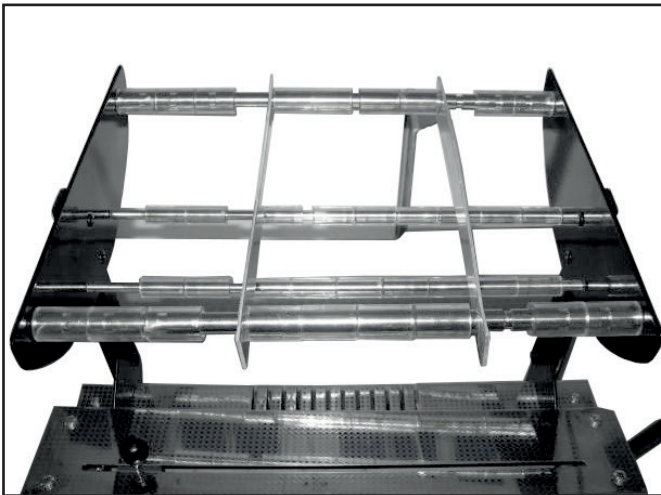**STEP 12**

Fit the reel separator brackets (12) into the large tubular supports at the top.

Operation

FIG. 1



Fit the reels of paper between the separator brackets so that the glossy side is visible. Pass the strip of paper from the reel underneath the press (SEE FIG.1).

FIG. 2

Set the blade knob fully to the left. Pull out the paper from the reel depending on the length of the instrument for packing. Close the press using the handle on the side of the device. Wait 10 seconds then use the blade to cut the piece from the reel. (SEE FIG.2)

FIG. 3

Place the instrument for packing in the pouch (SEE ILLUSTRATION). Turn the pouch round and position it with the open part in the press, then close again using the lever. Wait another 10 seconds and the pouch will be perfectly sealed.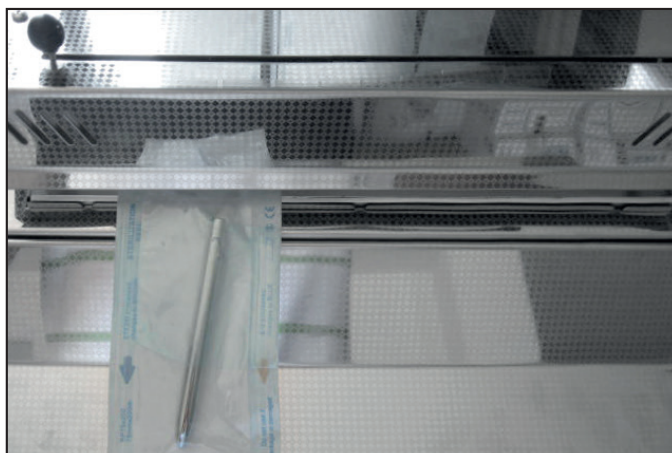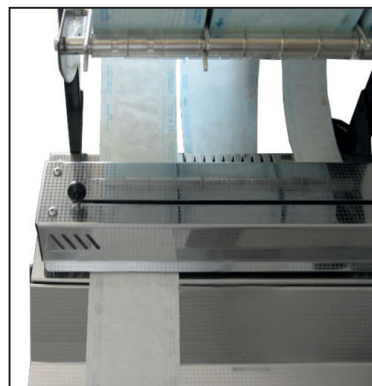

Technical characteristics

Model	INSTRUMENT PACKER
Supply Voltage	220-230V~ 50-60 Hz
Electrical Power	500 W
Weight	6 Kg
Dimension	mm 480X290X400 h


Cleaning

Always disconnect the plug from the power socket when cleaning the appliance.

Never use aggressive detergents or solvents.

Do not place in water.

To replace worn cutter blades unscrew the screws on the top of the press. SEE FIG. 1

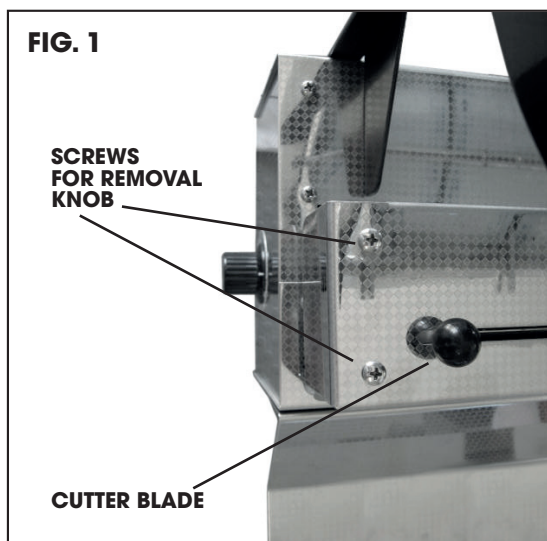Disposal

EU 2012/19/EU
(EN) This appliance complies with the EU
Directive 2012/19/EU.

The crossed out bin symbol on the appliance means that the product, at the end of its useful life, must be given to a differentiated waste collection centre for electric and electronic equipment or returned to the retailer when the delivery of the appliance to the proper collection facility at the end of its useful life. The appropriate differentiated waste collection for the treatment, recycling, and disposal of the appliance compatible with the environment contributes to prevent human health and favour the recycle of the materials the product is made of. The unauthorized disposal of the product by the user implies the application the Italian Legislative Decree n. 152/2006 (art. 255 and following amendments of the L. Decree n. 152/2006). For more detailed information about the available waste collecting systems, store where the purchase was made, buying a new equivalent appliance, since the product must be treated apart from the house waste. The user is responsible for possible negative effects on the environment and of administrative sanctions according to address to the local disposal service or the store where the purchase was made.

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XANITALIA Srl